WPulsePen rel. 2.3.2 software upgrade / [Aggiornamento del software WPulsePen rel. 2.3.2](#)**English**

The upgrade to the last release, for all the computers where the WPulsePen software is installed, it's highly recommended. We suggest to periodically visit the "Downloads" section of our website to verify the availability of new software updates/upgrades.

Rel. 2.3.2

Main changes compared with the release 2.3.1:

1. SEVR: the calculation algorithms were updated, the manually entered parameters can be saved in the archive, the complete projection of the SEVR was added.
2. Selective recalculation of markers: it is possible to choose which markers to recalculate (R, S / D, Sys).
3. R-detector improved and added the ability to invert the ECG signal.
4. Improved Ti detector and dicrotic notch (S / D) detector.
5. Changed the default of the PAM calculation which is now 40% instead of 33%.
6. Increased the number of cycles saved at low HR and increased the operating range of HR which now extends from 30 to > 300 bpm for ET and from 30 to > 400 bpm for 2T.
7. Added the export of the average curve including direct and reflected waves: activated with the right mouse button on the direct / reflected graph.
8. 3D graphics: visualization of signals in overlay or 3D mode. Fixed some problems with some graphics cards and 3D accelerator.
9. Export: with "CTRL + -" it is possible to choose the type of artery to export and select the "privacy" mode for which the names of the patients are replaced by the initials.
10. When saving a peripheral exam, the previously saved distances for the same type of exam of the same patient are automatically proposed again.
11. LP 2.3.2: horizontal scrolling of the signal enabled with the mouse, keeping pressed the left button, and/or with the finger (touchscreen monitor).
12. Added the management of synchronism with external DAQ systems.
13. Changed the format of the ASCII export file: 4 columns with elapsed time, E + T / 2T raw data, synchronism.
14. Correction of miscellaneous bugs.
15. Slight further graphic changes.

Warning: if you are migrating from a software revision earlier than 2.3.1 to the software revision 2.3.2, you need also to upgrade the firmware of the PulsePen modules: wRs1 (USB), wEc1 (ECG), and wTn1 (Tonometer).

If your currently installed software revision is 2.3.1, you don't have to upgrade the firmware.

Read the "readme.txt" file in the WPulsePen 2.3.2 Package before start installation.

Please contact DiaTecne s.r.l. - info@pulsepen.com to receive the complete install package with related instructions, or in case of doubts.

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